

User Guide

This guidance should be read in conjunction with the IFU supplied with the specific Breathing Circuit, the Flow Driver and Heater Humidifier

TRANSITION FROM NASAL INTERFACE TO FACE MASK

Step 1

Detach the nasal interface from the long limb and the PEEP T-piece.



Step 2

Connect the straight purple connector to the end of the long limb.



Step 3

Attach the PEEP T-piece to the long limb using the straight purple connector.



Step 4

Remove the red cap and connect a suitable facemask to the patient connection.



Step 5

Create a seal over the patient's mouth and nose to commence therapy.



User Guide

This guidance should be read in conjunction with the IFU supplied with the specific Breathing Circuit, the Flow Driver and Heater Humidifier

TRANSITION FROM FACE MASK TO NASAL INTERFACE

Step 1

Detach the face mask and replace with the red cap.



Step 2

Detach the PEEP T-piece from the end of the long limb and remove the straight purple connector.



Step 3

Connect patient end connector of your long limb to one limb of your patient interface.



Step 4

Attach the PEEP T-piece to the other limb of the patient interface. Ensure red cap remains in place.



Step 5

Attach the interface to the patient to commence therapy.
(see separate GFU for interface fitting)

